vitalpeak

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VITALPEAK BS 100

Baby scale


Thank you for purchasing the VITALPEAK product.

Safety when working with the device and use

- Read carefully the enclosed safety instructions.
- For proper handling of the product, read the latest version of the manual, which can be downloaded at: www.vitalpeak.eu.
- The device is intended for use in a domestic or private environment only, not in a commercial or medical field.
- The manufacturer is not liable for damage caused by improper or incorrect use.

Before use:

1. Open the battery compartment cover on the bottom of the device.
2. Insert four 1.5 V AAA batteries and replace the cover.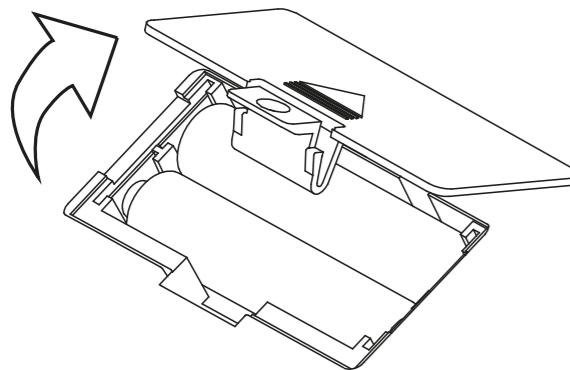

Using the device:

1. Always keep the scale on a firm, level surface, otherwise the weighing may be inaccurate.
2. To turn on the scale, briefly press the On / Off button on the device.
3. Press and hold the user button to select a user (switch between the users by a short press of the user button) the selected user is then confirmed by another long press of the user button. The user number is in the upper left corner of the LCD display next to "code".
4. Remove the silicone plug and insert the top board into the slot under the plug. Raise the measuring handle to activate the measuring mode. Leave the measuring board fully inserted. The display should show the value 46 cm or 18.1 in. If not, briefly press the On / Off button to reset the values. Then fully extend the measuring board.
5. Carefully place the baby on the scale. The head should touch the top board. Keep sliding in the measuring board until the feet touch the measuring handle.
6. Hold down the HOLD / UNIT button to freeze the values. The values will flash twice to confirm that they have been written to memory. Previous data will be displayed as a difference from the currently measured value. The up arrow on the right side of the display indicates a higher current measured value compared to the stored one, the down arrow indicates a lower current measured value compared to the stored one. Warning! When in Visitor mode the values will not be saved or compared. Briefly press HOLD / UNIT to freeze the values on the display.
7. The maximum number of values per user is 15. If this number of measurements is exceeded, the previous measurements will be gradually overwritten.
8. For the TARE (import) function, place an object on the scale that you do not want to include in the resulting weight and press the On / Off button. The scale resets and you can continue weighing.
9. Hold down the UNIT / HOLD button on the scale to change weighing units while the units are being changed.
10. Switching off the scale - hold down the On / Off button for a longer time. Otherwise, the scale switches itself off automatically one minute after the last interaction.


Error indication on the scale's display

Err – Overload The maximum allowable value of 25 kg / 55 lb has been exceeded.
Lo – Low batteries Replace batteries.

Specifications:

Units: kg / lb, cm / in
Power: 4x 1.5 V AAA batteries (not included)
Automatic shut-off: 2 minutes
Maximum load: 25 kg / 55 lb
Weighing deviation: ≤ 5 g, ≤ 0.01 lb
Height measuring range: 46–80 cm / 18.1–31.5 in
Measuring deviation: ≤ 1 mm, ≤ 0.1 in
Memory: 8 users, 15 values for each user
Visitor mode: Memory position 0 - measurements are not saved

LCD display description
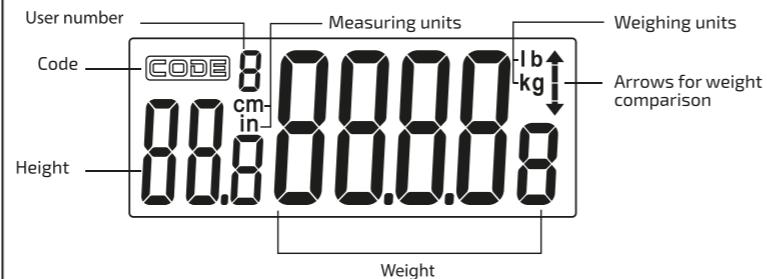


Buttons description

ZERO Button for switching On / Off and resetting the scale (TARE)

HOLD UNIT Button for recording, freezing and comparing the stored weight (press briefly) / changing the weighing unit (hold)

User button - Brief press displays the measurement history, scrolling through the individual values is done by a brief press. A long press on the button starts the changing user mode. Brief presses then switch between users. Confirm the selected user by press and hold.

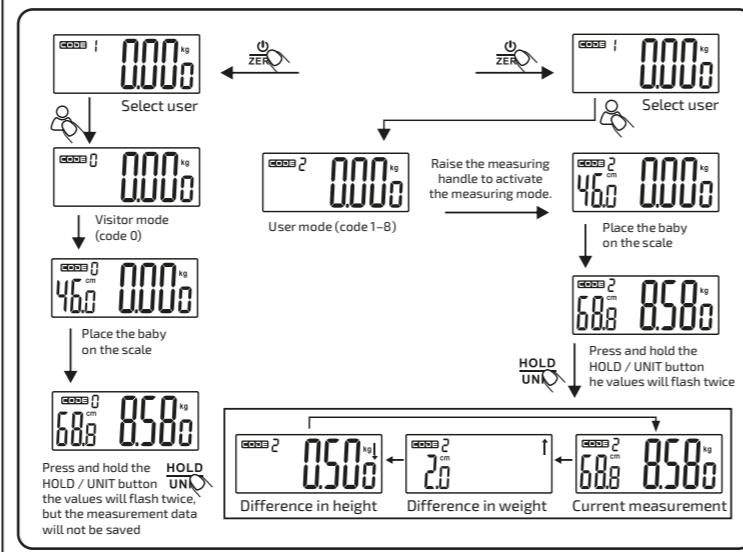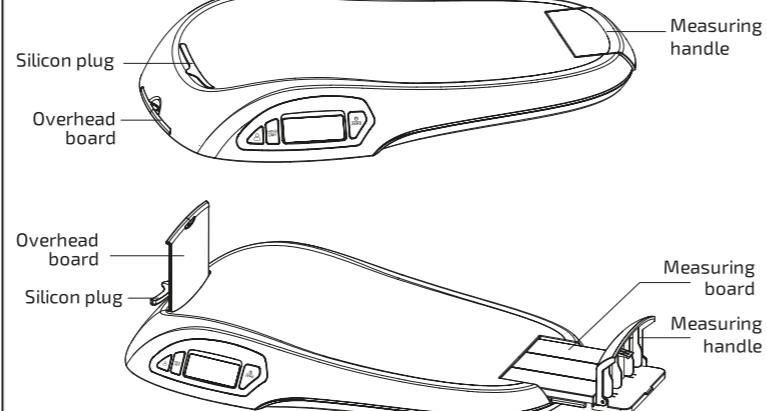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