vitalpeak

VITALPEAK HP 40

ENG INSTRUCTIONS FOR USE

VITALPEAK HP 40 Heated boot

Thank you for purchasing VITALPEAK

Safety precautions

- Read carefully enclosed safety instructions.
- To use the product properly, consult the latest version of the user’s manual available at www.vitalpeak.info.
- Never use the appliance folded and never cover the controller.
- Never use damaged appliance (damaged power cable and other damages - broken heating material, etc.)
- Never repair broken appliance. Contact the producer or the shop where you purchased the appliance.
- Do not break the product with sharp object or pin the appliance with them.
- If the controller flashes, it indicates it is not connected properly with the heating appliance. Turn the appliance off and check the connection of the controller with the heating appliance.
- Use the appliance under continuous supervision and never leave it on unkept. The appliance should not be used by children or persons with reduced physical, sensory or mental capabilities, without the need for an adult responsible for their safety.
- Do not use the appliance close to liquids that could spill out on the product. In such a case, immediately disconnect the appliance from the wall plug and clean it in accordance with the rules in the "Cleaning and Maintenance" article and have it dry completely.

Use of the product

- Make sure heating is set to 0 position on the controller.
- Check every part is connected properly – the connection between the controller and the heating appliance is to be firm.
- Connect the appliance in the wall plug.
- Select desired heating mode:
 - 0 - The appliance is off and it is not heating.
 - 1 - Low performance (recommended for a longer use).
 - 2 - Medium performance
 - 3 - High performance
- For repeated later use, switch the controller to 0 position and unplug the appliance from the grid.

Automatic switch off

- The appliance is equipped with the function of automatic switch off after 90 minutes of operation.
- If you want to use the appliance again, set the controller to the 0 position for 10 seconds, then switch the appliance on again. After you finish using the appliance, set the 0 position on the controller and unplug the appliance from the grid.

Cleaning and Maintenance

CAUTION! Always unplug the appliance from the grid before cleaning as there is the risk of electric shock.

- The controller must never be exposed to humidity or water. Clean only the heating appliance.
- Do not wash the heating appliance in the washing machine, spin-dry, dry in a dryer, wring, iron nor take it to the dry cleaners.
- Clean the appliance with a damp cloth or a sponge.
- The inner lining of the boot can be removed and washed in hand or in washing machine with a gentle program with maximum temperature of 30 °C.
- Never dry the appliance by switching it on.
- Use the appliance when it is completely dry.

Storage

- Always have the appliance cooled down before storage. Otherwise it may get damaged.
- Fold the heating appliance gently to avoid breaking. Never stack anything on the appliance when stored.
- Handle the supply cable carefully. Do not break it or wrap around the appliance.
- Store in cold, dry and safe environment out of reach of children.

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